

SmartDrive MX2+ | PushTracker E2

User's Manual



Wear OS by Google

Compatible with Android and iOS phones / devices.

Get the PushTracker app from the Google Play Store on your PushTracker E2 and start monitoring your wheelchair propulsion activity – **even if you don't have a SmartDrive!**
[see Using Your PushTracker E2]

To get the most out of your SmartDrive, download the PushTracker app from the Apple or Google Play app Store.



Check apps in the Store regularly for the latest updates!

Informational videos available at
www.permobilsmartdrive.com

permobil



Table of Contents

| | |
|--|----------|
| 1. Introduction | 3 |
| Intended Use of SmartDrive MX2+ App | 3 |
| Warnings and Safety Precautions | 3 |
| Electromagnetic Interference (EMI) | 5 |
| Bluetooth Wireless Communication | 5 |
| 2. Using Your PushTracker E2 | 5 |
| Attaching Your PushTracker E2 | 5 |
| Initial Set-Up | 6 |
| SmartDrive Pairing | 6 |
| Power Assist Activated | 6 |
| 3. Companion App | 7 |
| PushTracker Apps | 7 |
| PushTracker Wear OS App | 7 |
| PushTracker Mobile App | 8 |
| 4. Maintenance | 8 |
| Inspection | 8 |
| Updates and Other Maintenance | 8 |
| Troubleshooting | 8 |
| 5. Warranty | 9 |
| Manufacturer's Contact Information | 9 |
| 6. European Commission (EC) Authorized Representative | 9 |

1. Introduction

Intended Use of SmartDrive MX2+ App

The SmartDrive MX2+ software application (app) is intended to be used on the PushTracker E2 (TicWatch E2) to operate the SmartDrive Wheelchair Power Assist device.

Refer to the labeling and documentation of other components or product for their intended use, instruction, wireless, EMI and all other specification / information.

Warnings and Safety Precautions



All warnings and safety precautions issued by one's wheelchair manufacturer must be heeded in addition to those stated in this and any other provided manuals / instructions / guides, to include that provided within the SmartDrive MX2+ app. Failure to do so could result in serious injury or death.



Use of the SmartDrive MX2+ app or the SmartDrive by anyone who has not received proper instruction from this manual, the SmartDrive User's Manual and any other provided manuals / instructions / guides may result in serious injury or death.

Use of the SmartDrive MX2+ app is to be ONLY on the PushTracker E2 (TicWatch E2) through its release in the applicable app Store(s) as made available by Permobil [SmartDrive] / Max Mobility.

Always inspect your PushTracker E2 for missing parts / damage prior to each use.

Make sure your PushTracker E2 is charged before use.

When crossing major roads, intersection, railway crossings or highways as well as when you drive steep, long slopes you should always consider having somebody accompany you in the interest of your safety.

The PushTracker E2 should ONLY be worn by the user sitting in the wheelchair with the SmartDrive attached.

Always turn OFF the power assist [via SmartDrive MX2+ app] when parked to prevent accidental activation.

Always turn OFF the power assist [via the SmartDrive MX2+ app] before taking the PushTracker E2 off of your wrist.

(continued on next page...)

Tapping force does not need to be excessive or "hard". Gentle, stiff wrist taps at a reasonable speed [i.e. one half (0.5) second between taps] is sufficient. Refer to the SmartDrive User's Manual for complete instruction.

Always use caution when transferring in or out of the wheelchair. Every precaution should be taken to reduce the transfer distance and be certain the wheel locks are engaged to prevent the wheels from moving. Also always make sure the power assist is OFF [via the SmartDrive MX2+ app] when transferring to and from your chair.

The PushTracker E2 has passed industry climate testing, and is considered swim-ready [up to 50 m]. Refer to the other provided manuals / instructions / guides for complete detail regarding its water resistance performance.

Ensure your device is running the latest versions.

NOTES: Google Play, Wear OS by Google, Android, the Apple logo, iPhone, iOS and Bluetooth are trademarks or registered trademarks of their respective owners.

Access to the Google Play Store to get the SmartDrive MX2+ [and any other] app on the PushTracker E2 may require the use of an existing Google account or creation of a new one. Separate policies, terms and conditions apply.

Figures and diagrams in this manual may vary slightly from your device.

All product [including components], requirements, policies, terms and conditions, to include the contents of this and any other provided manual / instruction / guide, are subject to change without prior notice. For current product instruction and information refer to the official website.

An electronic version of this User's Manual along with informational videos are available on our website [www.permobilsmartdrive.com] for people with visual, reading or cognitive disabilities.

The Serial Number of your PushTracker E2 can be found in the Settings app.

It is recommended that you save this User's Manual and record the Serial Number below for reference.

Device Serial Number: _____

Electromagnetic Interference (EMI)

Refer to the **SmartDrive User's Manual** for EMI specifications.

Bluetooth Wireless Communication

Bluetooth wireless communication [2.4 GHz frequency band] is used by the PushTracker E2 to communicate to the SmartDrive when to activate/deactivate the power assist, send and receive signals from a smartphone, etc. The PushTracker E2 must be within 25 ft (8 meters) of the SmartDrive / smartphone to connect and communicate. The maximum output power of this technology is 10 mW, and the operation of this device in close proximity to other wireless devices [i.e. electromagnetic security systems, Bluetooth headsets, etc.] could cause the device to lose connection. Power assist is always deactivated whenever the motor is driving and there is a loss of connection; the SmartDrive will beep and PushTracker E2 will also give indication when this happens. Worst case coexistence testing was performed on the device to ensure its safety and effectiveness. Please contact Permobil [SmartDrive] / Max Mobility immediately for troubleshooting if continuous connection issues are experienced with the device.

2. Using Your PushTracker E2

Attaching Your PushTracker E2

The PushTracker E2 is designed to comfortably and securely attach to the top part of wrists of all sizes. It has an embedded accelerometer to recognize when you push, when you want to activate the power assist and also when you tap your hand against the handrim to deactivate the power assist. Additionally, it is equipped with an AMOLED screen and vibration motor to indicate different information, states and activations / deactivations that occur during everyday use.

The band on the PushTracker E2 is designed for most users; but if you have difficulty securing the buckle / clasp, it doesn't fit right, or you wish to have one of a different color / material, the band can be easily replaced with essentially any 0.87 in (22 mm) wide off-the-shelf watch band of your choosing.



It is recommended that your PushTracker be worn snug on your wrist to get the most reliable recognition of your hand movement for SmartDrive operation, propulsion activity tracking, etc. Ability to operate / maneuverer the wheelchair can be affected if the PushTracker, watches, jewellery, etc. are worn loosely or positioned such that they can become loose and interfere with handrim operation.

Initial Set-Up

After powering on, follow the prompts closely on the PushTracker E2 to initially set up with your smartphone / device. If any serious difficulties are experienced during set-up, go to the Permobil [SmartDrive] / Max Mobility website for more information and support.



A Wi-Fi connection on the PushTracker E2 may be required to access the Google Play Store. If you do not have access to a Wi-Fi network in your location, set up and connect to a Personal Hotspot from your smartphone / device. Carrier or device restrictions and data rates may apply.

[Skip if you do not have a SmartDrive] Upon successful completion of set-up, go to the Google Play Store app on the PushTracker E2, search for [i.e. "smart drive", "smartdrive", etc.] and get the SmartDrive MX2+ app.



SmartDrive MX2+

To start monitoring your wheelchair propulsion activity **even if you do not have a SmartDrive**, go to the Google Play Store app on the PushTracker E2, search for [i.e. "push tracker", "pushtacker", etc.] and get the PushTracker app [see the "Companion App" section for more details].

SmartDrive Pairing

Initial pairing of the PushTracker E2 to a SmartDrive is required, and this is done in the SmartDrive MX2+ app. If the SmartDrive MX2+ app has not been previously paired to a SmartDrive, turning power assist ON will initiate the pairing. If the SmartDrive MX2+ app has been previously paired to a SmartDrive, pairing can be done in the Settings menu on the app.


Power Assist Activated

After successful pairing, power assist is turned ON and the SmartDrive MX2+ app is connected to the SmartDrive, the Indication will flash blue. When the SmartDrive motor is driving, the Indication will be solid blue. The Indication will flash red when

the SmartDrive MX2+ App is trying to connect or has lost wireless connection to the SmartDrive. Whenever there is a change of state to the SmartDrive, you will feel vibration as additional indication from your PushTracker E2.

When power assist is ON and the SmartDrive motor is driving, the SmartDrive MX2+ app will display your current speed. The Indication will light up with various colors to notify you of the SmartDrive's current state.

Additional detail / information that is helpful in explaining different features of the app and the SmartDrive is provided throughout the SmartDrive MX2+ app [by clicking (i)]. Refer to the **SmartDrive User's Manual** for its operation information and instruction.

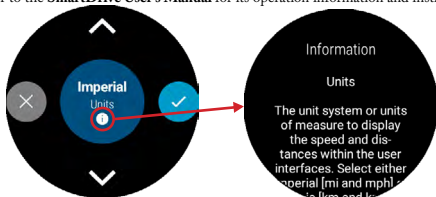

3. Companion App

PushTracker Apps

For the best experience with your SmartDrive and PushTracker E2, download the PushTracker app on both your PushTracker E2 and your smartphone. This will allow you to track your propulsion activity progress, set goals and more.

PushTracker Wear OS App



Within the SmartDrive MX2+ app there is an option to connect to the PushTracker mobile (smartphone) app. The first time this is selected you will be taken to the Google Play Store to get the PushTracker app on your PushTracker E2 [if this has not already been installed]. This app is required on the PushTracker E2 in order to use the PushTracker mobile app.

PushTracker Mobile App



Download the PushTracker mobile (smartphone) app on your phone / device to get an enhanced view of your propulsion activity. In this app you will also have the added functionality of setting your activity goals, track your progress and more.

4. Maintenance

Inspection

Always inspect the PushTracker E2 for missing parts / damage prior to each use. Cease use and contact Permobil [SmartDrive] / Max Mobility or your mobility equipment dealer if you notice any of the following:

- Broken or cracked plastic
- Broken buckle or ripped bands
- Damaged display

Updates and Other Maintenance

Check regularly to ensure your device is running the latest versions, as features are added and improvements are made.

Troubleshooting

Contact Permobil [SmartDrive] / Max Mobility or your mobility equipment dealer for any other issues that you experience with the device or app. It is strongly recommended to update the versions on your device to the latest version prior to any troubleshooting as your issue may have been resolved in later releases.



Li-ion

Do not throw away the PushTracker E2. Please properly dispose of the battery by recycling at the end of its life or send it back to Permobil [SmartDrive] / Max Mobility.



5. Warranty

Permobil [SmartDrive] / Max Mobility offers a 1 year warranty against manufacturing defects. Permobil [SmartDrive] / Max Mobility's obligation under this warranty shall be limited to repairing or providing replacement of part(s), which are found to be defective. Any part(s) of the system replaced by us at our discretion shall be with a functionally operative part.

This warranty may be voided if: 1) the product is not used according to instructions given in this instruction manual, 2) defects were caused by improper use as determined by the company personnel, 3) modification or alteration of any nature is made in the electrical circuitry or physical construction, or 4) defects are due to cause beyond control like lightning, abnormal voltage or while in transit at the purchaser's place of business.


Manufacturer's Contact Information

Manufacturer: Max Mobility, LLC
Address: 300 Duke Drive
Lebanon, TN 37090
USA
Phone: (615) 953-5350 / Toll-Free (800) 637-2980
Fax: (888) 411-9027
Website: www.permobilsmartdrive.com / www.pushtracker.com
Email: support.smartdrive@permobil.com

6. European Commission (EC) Authorized Representative



Advena Ltd.
Tower Bus. Ctr., 2nd Floor
Tower Street
Swatar BKR 4013
Malta

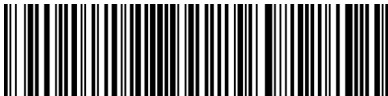

MX2E2-USM Rev. A